

## Care and Maintenance

- Wash in cold water, gentle cycle and mild soap
- Do not dry clean or tumble dry - allow to dry naturally
- Do not use bleach



## Ventilated Lumbar Sacral Support

### Fitting Guidelines & Care Instructions



VLSS IFU - Revision 2 - Jan 2024

**CHANECO LTD**  
4 Kilvey Road, Brackmills, Northampton, NN4 7BQ  
T: 01604 709999 F: 01604 700057  
[www.chaneco.co.uk](http://www.chaneco.co.uk)

Chaneco Limited is registered in England under Company Number 11511502. Registered office as above.



Single Patient Use



Not Fire Retardant



## Features

- Constructed of vented, breathable, elastic stretch material
- Depth 10" (25cm) back and 7½" (18cm) front
- Durable elastic double fulcrum pulls for additional compression and support
- Four rigid stays provide firm circumferential support
- Secure touch and close fastening

## Indications

- Low back pain
- Lumbar sprains and strains
- Post-trauma/surgery
- Patients who find conventional corsets unpleasant to wear

## Size Table

Size	Waist Circumference		Code
	Inches	CM	
Small	26 - 30	56 - 71	VLSS-S
Medium	30 - 34	69 - 84	VLSS-M
Large	34 - 38	81 - 96	VLSS-L
X Large	38 - 42	94 - 109	VLSS-XL
XX Large	42 - 46	107 - 122	VLSS-XXL

## Fitting Instructions

- Open out the abdominal support
- Position the support around your waist
- Wrap the support around your waist and secure using the hook and loop fastening



## Additional Information

- Material content:: Nylon 38%, Rayon 34%, Rubber 28%
- CONTAINS LATEX